


LIGASANO®

CE0481

Art.-No. 15090-012

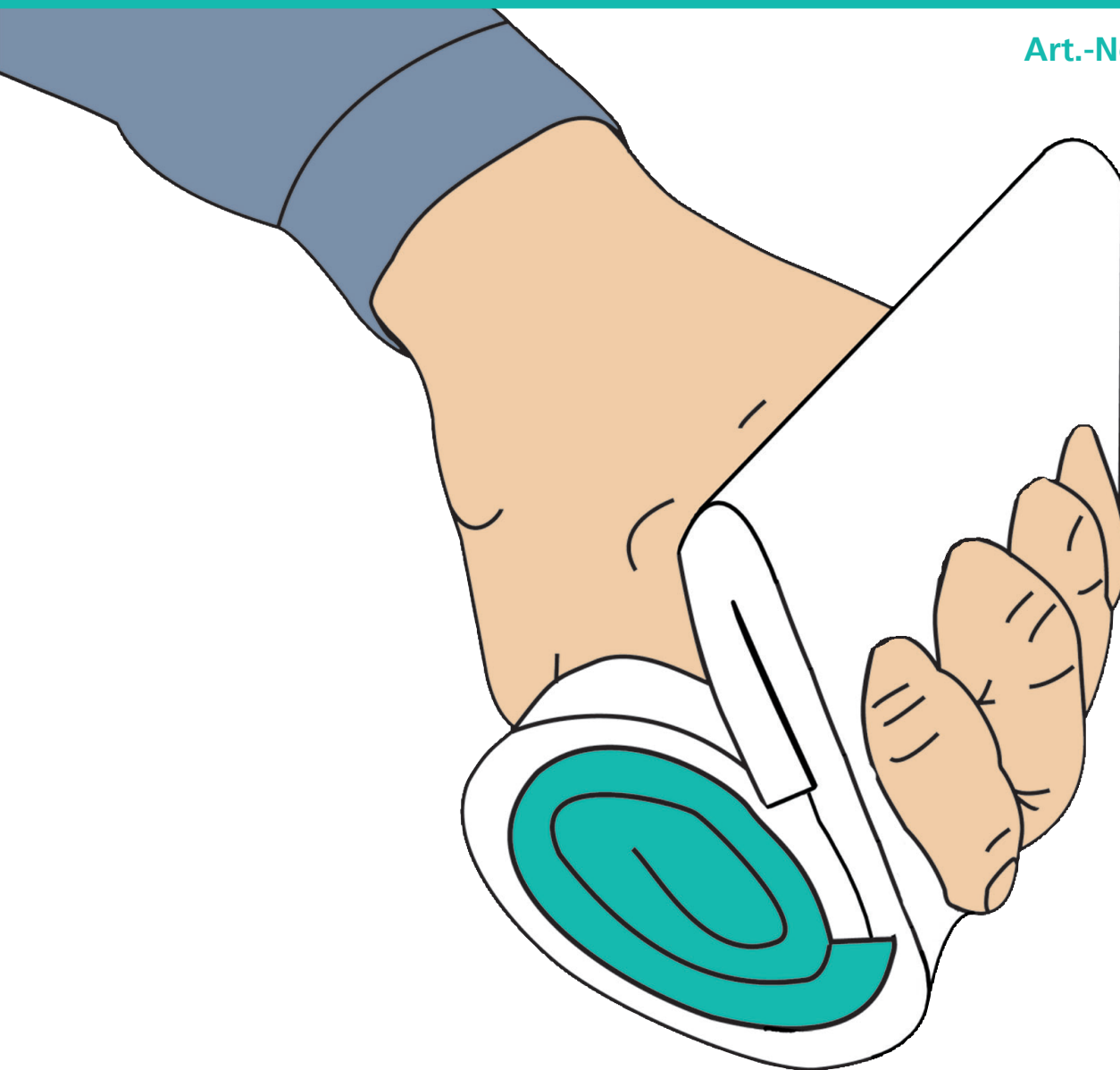

Palm Dressing

made from LIGASANO® white and LIGASANO® green

LIGASANO®
Simply Works

Content:

- 2 pcs. Spiral rolls made of LIGASANO® green, diameter approx. 8cm, width 10cm
- 10 pcs. Palm dressing made of LIGASANO® white, 29x12x1cm

1

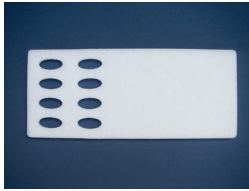


Image 1:

The palm-dressing made of LIGASANO® white has two rows each with four holes for the fingers.

2



Image 2:

Fold over and bring the two rows together. Now place your fingers through the aligned holes in both layers.

3



Image 3:

Please note: the shorter end always has to lie against the palm.

4



Image 4:

The palm dressing can prevent or treat intertrigo of the skin and contracture injuries of the palm and fingers by keeping space between the surfaces of the skin. The mechanical stimulus of the LIGASANO® white on the skin promotes local blood flow and removes unviable skin cells.

5



Image 5:

Additionally you can use a spiral roll of LIGASANO® green to get greater resistive pressure or even to open slightly to provide less counter pressure. Due to the special properties of the LIGASANO® green accumulation of heat or moisture is prevented. **If only the contact to the skin of the palm may cause spasticity, this application is contraindicated.**

6



Image 6:

You can adapt the diameter as well as the compressive strength of the LIGASANO® green spiral roll. If you open the roll and coil it up in a tight, small diameter, greater strength results. If a smaller diameter with less strength is preferred, you can cut a part of the outer roll off.

7



Image 7:

In the case of complete hand contractures you can utilize and adapt the LIGASANO® hand dressing easily and benefit from the advantages of avoiding the skin-to-skin contact.

8



Image 8:

An efficient fixation can be made with LIGAMED®*fix*.

The palm-dressing is made of LIGASANO® white and is a disposable product that loses its properties after a few days. Unless the dressing contains infected material, dispose of the dressing with the household waste.

The spiral rolls made of LIGASANO® green are usable for a longer time (e.g. for a hand workout), are washable up to 95°C and sterilisable with steam (if required please ask for our special information sheet).