
# LIGASANO®

Art.-Nr. 15090-012

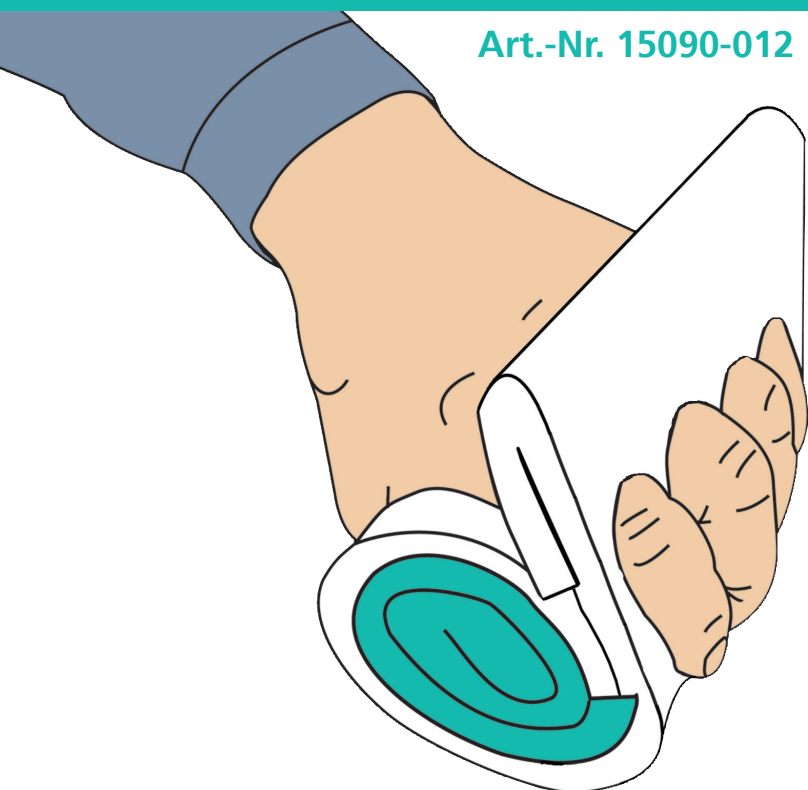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**Image 1:**

The palm-dressing made of LIGASANO® white has two rows each with four notches for the fingers.

**Image 2:**

By bending down you can bring the two rows one upon the other. Now stick the fingers through the both layers.

**Image 3:**

Please note: the shorter end always has to lie towards the palm.

**Image 4:**

By this you can treat or prevent intertrigo, dents and hurts of the palms and between the fingers. The mechanical stimulus, which effects the LIGASANO® white to the palm, is favouring the local blood flow; non vital dermal cells peel away.

**Image 5:**

Additionally you can insert a spiral roll of LIGASANO® green to get a workout effect or even to open it a little bit once again by permanent light counter pressure. Due to the special properties of the material you can prevent accumulation of heat or moisture. If only the contact to the skin of the palm may cause spasticity, this application is contraindicated.

**Image 6:**

You can adapt the diameter as well as the strength of the LIGASANO® green spiral roll. If you open the roll and coil it up narrower, a smaller diameter with higher strength results. If a smaller diameter without higher strength is favoured, you can cut a part of the outer coat.

**Image 7:**

Also in the case of completely closed hands you can utilize and adapt the LIGASANO® palm dressing easily, with all the known advantages in avoiding the skin-to-skin contact.

**Image 8:**

An efficient fixation can be made with LIGAMED®/x.

**The palm-dressing made of LIGASANO® white is a disposable product, that loses its properties after a few days. Except for infected dressings, you can dispose it with the household waste. The spiral rolls made of LIGASANO® green are utilizable for a longer time (e.g. for a workout of the hands), are washable up to 95°C and sterilizable with steam (if required please ask for our special information brochure).**

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